


STEP BY STEP DIRECTIONS FOR USING THE NAPA LP-15 AIRWAY PRESSURE MONITOR WITH BCPAP



STEP 1: Press the On/Off button to turn the NAPA LP-15 Airway Pressure Monitor on.

STEP 2: Use the following diagram to set high and low alarm limits and adjust alarm volume:

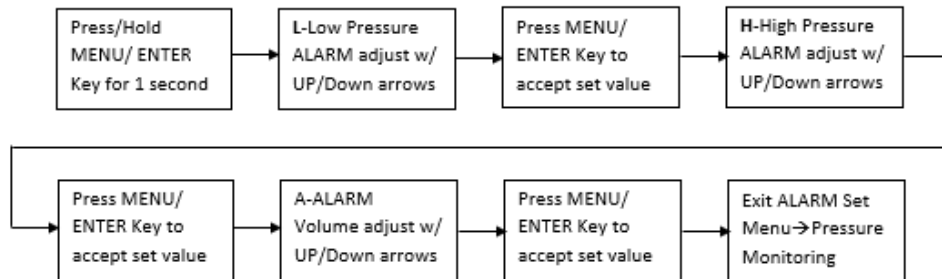

NOTE: Be aware there is a two-minute delay prior to alarm activation following power-up.

STEP 3: Attach the twist lock connector end of the pressure tubing assembly to the fitting on the bottom of the NAPA LP-15 Airway Pressure Monitor.

STEP 4: Locate the pressure fitting in the patient's breathing circuit.

STEP 5: Connect the luer connector end of the pressure tubing assembly to the luer fitting in the breathing circuit.

STEP 6: The NAPA LP-15 Airway Pressure Monitor will now display mean airway pressure.

PLEASE NOTE: *The monitor is self-zeroing and the Zero Calibration button is unnecessary during BCPAP therapy. Should any questions arise about the displayed pressure, start by disconnecting the monitor and turning it off, and then powering up again and reconnecting. Calibration can be checked by performing a Functional Test as described on pg. 38 of the User Guide.*

